



Configura[®] Advance[™]

Information & assessment form



Local Provider: Emprise Mobility

Perth: 5 Sobek Pass | Mandurah: 14/8 Magenta Terrace
1300 761 196 | www.emprise.com.au



User Group

Configura® Advance is appropriate for:

- Non-ambulant users
- Users with limited knee extension
- Users who need to be transported from room to room
- Moderate postural requirements
- Users up to 160 kg
- Complex postural requirements, e.g. fixtures and contractures
- Users with stiffness or fixtures at the hips
- Stroke and neurological conditions

Configura® Advance is not appropriate for:

- Users who need a riser function to transfer
- Bariatric users

Configura® Advance intended environment:

- Homecare use
- Nursing facilities
- Hospitals



Prescriber details

Name: _____
Health Profession: _____
Address: _____

Telephone: _____
Email: _____

User details

Name: _____
Address: _____

Telephone: _____
Email: _____
User weight: _____
Pressure care: _____
Posture: _____
Mobility: _____
Diagnosis: _____
Prognosis: _____
Transfer method: _____
Current seating: _____
Problems with current seating: _____
Notes: _____

Configura Advance



Width adjustment from
350 mm to 500 mm



Dropdown armrest option -
helps with transfers and sling access



Legrest elevation with 37 degrees
of tilt-in-space



7 degrees of forward tilt



'Reduced shear' backrest
recline





Depth adjustment from
400 mm to 550 mm



Integrated self-supply overlay



Continuous seat overlay
for thigh support



Angle adjustable footplate




Height adjustable footplate






Size

Key thoughts:

- Three measurements are required for selecting a chair: seat depth, seat height and seat width – see drawings below for clarification.
- When selecting a chair size, prioritise the chair measurements in the following order: seat depth, seat height then seat width. This will ensure the selected size is the right overall proportion for the user.

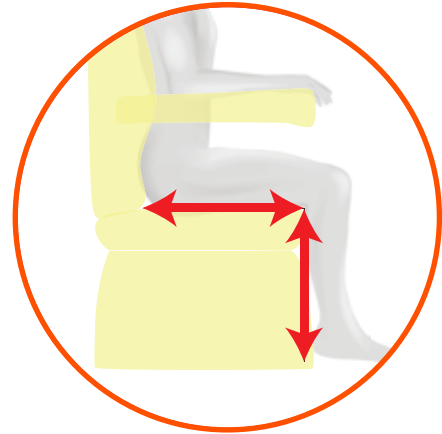

Chair size options

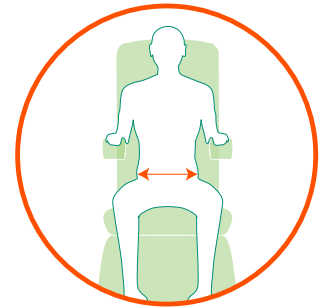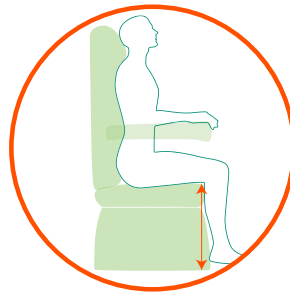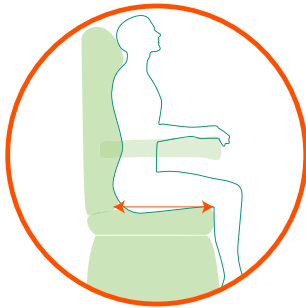

SEAT DEPTH	
422 mm (16.5") Pos. 1	<input type="checkbox"/>
438 mm (17.5") Pos. 2	<input type="checkbox"/>
454 mm (18") Pos. 3	<input type="checkbox"/>
470 mm (18.5") Pos. 4	<input type="checkbox"/>
486 mm (19") Pos. 5	<input type="checkbox"/>
502 mm (20") Pos. 6	<input type="checkbox"/>
518 mm (20.5") Pos. 7	<input type="checkbox"/>
534 mm (21") Pos. 8	<input type="checkbox"/>
550 mm (21.5") Pos. 9	<input type="checkbox"/>

SEAT HEIGHT <small>*position of footrest on leg-rest</small>	
325 mm (13")	<input type="checkbox"/>
350 mm (14")	<input type="checkbox"/>
350 mm (14")	<input type="checkbox"/>
400 mm (16")	<input type="checkbox"/>
425 mm (17")	<input type="checkbox"/>
450 mm (18")	<input type="checkbox"/>
475 mm (19")	<input type="checkbox"/>
500 mm (20")	<input type="checkbox"/>
525 mm (21")	<input type="checkbox"/>

SEAT WIDTH	
350 mm (14") Only available with dropdown armrests.	<input type="checkbox"/>
400 mm (16")	<input type="checkbox"/>
450 mm (18")	<input type="checkbox"/>
500 mm (20")	<input type="checkbox"/>
550 mm (22") As standard this will not go through a 792mm door. Please use selection box below if you require narrower arms.	<input type="checkbox"/>
550 mm (22") with narrow arms Only required if the chair has to go through a 792mm door.	<input type="checkbox"/>

SEAT HEIGHT	
609mm (24") (standard)	<input type="checkbox"/>
533mm (21") (lower frame mounting)	<input type="checkbox"/>

Pressure

Key thoughts:

- Selection of pressure cushions is a clinical decision which involves many variables including undertaking a risk assessment. Professional input from a Wound Nurse may be required at this stage.
- The Advance chair features vapor-permeable fabric as standard on seat base, legrest, backrest and armrests.



Pressure care options

DESCRIPTION	INFORMATION	IMAGE	SELECT
Visco cushion (standard)	<p>Prevention: Heat-sensitive memory foam molds to the user, providing maximum support and pressure reduction. Suitable for users who are at risk of pressure damage.</p> <p>160kg (25 stone) weight capacity.</p>		<input type="checkbox"/>
Supply own cushion	<p>Cut-out allows integration of self-supply cushion of approximate dimensions 438mm (17.5") x 438mm (17.5") x 75mm (3")</p> <p>See page 12 for instructions.</p>		<input type="checkbox"/>
Portable Cushionair Cushion Kit	<p>Management: Powered alternating air cushion which provides excellent comfort, stability and pressure relief. The pump is battery powered.</p> <p>160 kg (25 stone) weight capacity.</p> <p>Code: CA2710</p>		<input type="checkbox"/>

Posture

Key thoughts:

- Tilt-in-space can significantly improve midline sitting, potentially negating the need for additional lateral support where a user tends to slump mildly to one side.
- Filling can be removed from the multi-adjustable pillow backrest to accommodate different postural requirements.



Backrest options

The Advance chair has backrest recline, with a label showing the angle. There is also an option to lock the backrest at a particular angle. This needs to be reviewed when the chair is set up.

Option one

STANDARD MULTI-ADJUSTABLE BACKREST	SELECT
	<input type="checkbox"/>

Option two

POSTURAL BACKREST	SELECT
	<input type="checkbox"/>

TOP PILLOW SETTINGS



LEFT WING SETTING

1 2 3 4 5 6

PILLOW ANGLE

1 2 3 4 5 6

RIGHT WING SETTING

1 2 3 4 5 6

PILLOW DEPTH

1 2 3 4 5 6

MIDDLE PILLOW SETTINGS



LEFT WING SETTING

1 2 3 4 5 6

RIGHT WING SETTING

1 2 3 4 5 6

PILLOW DEPTH

1 2 3 4 5 6

BACKREST HEIGHT

1 2 3 4 5 6 7 8






Backrest cushions:

All the backrest cushions can have filling removed to help with comfort and accommodating the user's needs.




Postural Accessories

DESCRIPTION	LONG DESCRIPTION	PART NUMBER	IMAGE	
Lateral Supports	<p>Accomodation: used under the Multi-adjustable Pillow Backrest cushion to provide moderate postural support</p> <p>Sold as each.</p>	CA2413		QTY 1 <input type="checkbox"/>
				QTY 2 <input type="checkbox"/>
Cocoon Backrest	<p>Encouragement: encourages a good midline sitting position for users whose posture is flexible.</p>	CA2711		S <input type="checkbox"/>
				M <input type="checkbox"/>
				L <input type="checkbox"/>
Adjustable Lateral Support Backrest	<p>Management/Correction: contoured foam backrest with adjustable Velcro Lateral Supports. Provides more aggressive lateral support and mild positioning/correction. Supplied with breathable, pressure reducing fabric.</p>	Small CA2410 Medium CA2411 Large CA2412		S <input type="checkbox"/>
				M <input type="checkbox"/>
				L <input type="checkbox"/>
50mm (2") Depth Adjustment Kit	Decreases depth by further 51mm (2").	CA2416		<input type="checkbox"/>

Extras

DESCRIPTION	LONG DESCRIPTION	PART NUMBER	IMAGE	
Four-point Pelvic Positioning Belt	Pelvic strap to act as a prompt to aid pelvic positioning.	Code: CA2708		<input type="checkbox"/>
Electric Operation	Battery-operated electric control of tilt-in-space and legrest elevation.	Coming Soon		<input type="checkbox"/>
Dropdown Armrests	Drop armrest to facilitate side transfers and sliding access. Three armrest height settings - refer to IFU for adjusting. Fitted as standard on chair if 350 mm width is selected.	See parts list and codes on page 11		6" <input type="checkbox"/>
				7" <input type="checkbox"/>
				8" <input type="checkbox"/>
Profiled Headrest	Suitable for users with reduced head control. Fits over the shoulders.	Code: CA2706		<input type="checkbox"/>
External Lateral Supports	These are only compatible with the postural backrest and will be required when managing more complex postural requirements.	Code: CA2705		<input type="checkbox"/>
Tray	Full-width tray that is positioned over the armrests.	Code: CA2707		<input type="checkbox"/>

Other adjustments

DESCRIPTION	LONG DESCRIPTION	IMAGE	
Forward tilt	This is standard on the electrically operated chairs. Forward tilt is available on the manually operated chairs as a no-cost option. A risk assessment must be carried out if this option is to be used.		<input type="checkbox"/>
Backrest recline	Standard on the chair and can be adjusted following chair delivery.		<input type="checkbox"/>
Footplate angle	Footplate comes as standard on the chair. It is angle-adjustable and can be adjusted following chair delivery.		<input type="checkbox"/>

Specific dimension

DESCRIPTION	VALUE
Seat height	533mm (21") - 609mm (24")
Seat width	406mm (16") - 558mm (22")
Seat depth	406mm (16") - 558mm (22")
Footplate height adjustment	330mm (13") - 533mm (21")
Armrest height	203mm (8") as standard - adjustable with dropdown armrest
Overall width	685mm (27")
Overall height - upright	1,320mm (52")
Overall length - without footplate	939mm (37")
Overall length - with footplate	1,295mm (51")
Total length fully tilted, reclined and legrest elevated	1,879mm (74")
Tilt-in-space range	7 - 37 degrees
Forward tilt	10 degrees
Backrest recline	130 degrees

Advance parts and compatibility

DESCRIPTION	CHAIR WIDTH SETTING				
	14" (35.56 cm)	16" (40.64 cm)	18" (45.72 cm)	20" (50.8 cm)	22" (55.88 cm)
Standard Armrests (excluding metal brackets)	NA	Code: SC2568			
Dropdown Arms - 16" (41cm) - 22" (56cm)	NA	Code: CA2702			
Dropdown Arms - 14" (36cm) - 20" (51cm)	CODE: CA2700				NA
Dropdown Arms - 18" (46cm) - 22" (56cm)	NA	NA	CODE: CA2701		
Dropdown Arms frame and cover kit. This is not included in the above	CODE: SC2570				
Seat overlay and integrated visco cushion	CODE: SC2566				
Postural Backrest	CODE: CA2709				
External Lateral Supports	CODE: CA2705				
Profiled Headrest	CODE: CA2705				
Tray	CODE: CA2707				
Footplate Kit	CODE: SC2561				


Cleaning instructions for the fabric

The vinyl fabric can be cleaned by applying a suitable cleaning agent from a trigger spray, in a fine mist. Agitate with a synthetic sponge and wipe off.

Interchangeable pressure cushion


The integrated pressure cushion is supplied as standard on all chairs. This allows for easy integration of alternative cushions. The size of the aperture is 430 mm x 430 mm x 76 mm (17.5" x 17.5" x 3").

Using a cushion of this size will ensure the height of the seat and armrests remains unchanged.

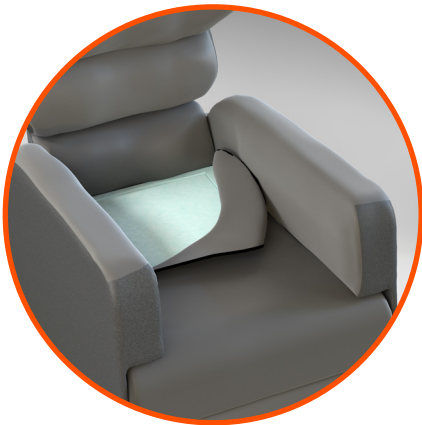


STEP 1

Remove the backrest cushions and unzip the cushion cover to expose the foam.

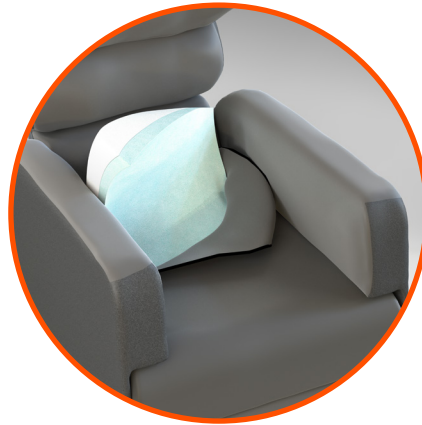

STEP 2

Pull the foam out carefully, starting at the rear and cut the perforations at the front.

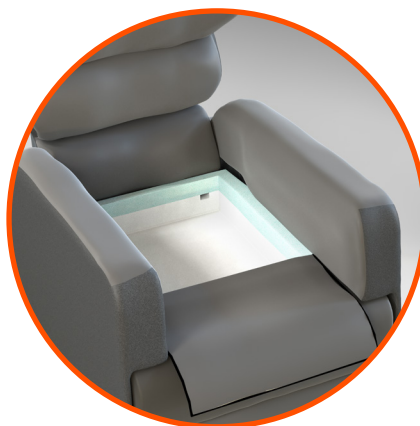

STEP 3

This leaves the cut-out for an alternative cushion to be inserted. Insert the cushion and rezip the cover.

NOTES



At Enable we source quality, affordable and innovative products globally and have them available for fast delivery. But we do more than just provide products.

We give the knowledge and support to help our customers grow their business and deliver better life care.



ENABLE LIFECARE PTY LTD

T 1300 370 370

enablelifecare.com.au